

Location Code

Different locations across the world have varying gravitational pulls. This in turn affects the accuracy of weight readings on scales. By selecting the correct gravity setting on your scale according to your geographical location, you are guaranteed the most accurate weight readings. The scale's default gravity setting is Location Code A2.

See the instruction manual to set the location.



Country	City	Location Code
Albania	Tirana	A4
Andorra	Andorra la Vella	A4
Armenia	Yerevan	A4
Austria	Wien	A3
Azerbaijan	Baku	A4
Belarus	Brest, Gomel, Hrodna, Mahilyow, Minsk	A2
Belgium	Bruxelles	A2
Bosnia and Herzegovina	Sarajevo	A4
Bulgaria	Burgas, Plovdiv, Sofia, Varna	A4
Croatia	Zagreb	A3
Cyprus	Nicosia	A5
Czech Republic	Praha Brno	A2 A3
Denmark	Copenhagen	A1
Estonia	Tallinn	A1

Country	City	Location Code
Finland	Oulu, Tampere Helsinki, Turku	A0 A1
France	Bordeaux, Lyon, Nantes, Paris, Strasbourg Marseille, Toulouse	A3 A4
Georgia	Tbilisi	A4
Germany	Berlin, Bremen, Düsseldorf, Frankfurt, Hamburg, Hannover München, Stuttgart	A2 A3
Greece	Athina	A5
Hungary	Budapest	A3
Iceland	Reykjavik	A0
Ireland	Dublin	A2
Italy	Milano Napoli, Roma Palermo	A3 A4 A5
Kazakhstan	Astana Almaty	A2 A4
Kosovo	Pristina	A4
Kyrgyz	Bishkek	A4
Latvia	Riga	A1
Liechtenstein	Vaduz	A3
Lithuania	Vilnius	A2
Luxembourg	Luxembourg	A3
Former Yugoslav Republic of Macedonia	Skopje	A4
Malta	Valletta	A5
Moldova	Kishinev	A3
Monaco	Monaco	A4
Montenegro	Podgorica	A4
Netherlands	Amsterdam, Rotterdam	A2
Norway	Hammerfest, Tromsø, Trondheim Oslo, Stavanger	A0 A1
Poland	Gdańsk, Kraków, Poznań, Warszawa, Wrocław	A2
Portugal	Lisbon, Porto	A4
Romania	Bucharest, Cluj-Napoca	A3
Russia	Moskva, Sankt-Peterburg	A1
San Marino	San Marino	A4
Serbia	Belgrade	A3
Slovak Republic	Bratislava	A3
Slovenia	Ljubljana	A3
Spain	Barcelona, Madrid, Valencia, Zaragoza Málaga, Sevilla	A4 A5
Sweden	Gävle, Umeå Göteborg, Malmö, Stockholm	A0 A1
Switzerland	Bern, Geneva, Zürich	A3
Tajikistan	Dushanbe	A4
Turkmenistan	Ashgabat	A5
Ukraine	Kiev L'viv, Mykolayiv	A2 A3
Uzbekistan	Tashkent	A4
United Kingdom	Aberdeen, Edinburgh, Glasgow Belfast, Birmingham, Liverpool, London, Manchester	A1 A2
Vatican	Vatican	A4

Location Code



Country	City	Location Code
Afghanistan	Kabul	A5
Algeria	Algiers	A5
Angola	Luanda	A8
Bahrain	Manama	A7
Benin	Porto-Novo	A9
Botswana	Gaborone	A7
Burkina Faso	Ouagadougou	A8
Burundi	Bujumbura	A9
Cameroon	Yaounde	A9
Cape Verde	Praia	A8
Central African Republic	Bangui	A9
Chad	N'Djamena	A8
Comoros	Moroni	A8
Congo (ROC)	Brazzaville	A9
Congo (DRC)	Kinshasa	A9
Cote d'Ivoire	Yamoussoukro	A8
	Abidjan	A9
Djibouti	Djibouti	A8
Egypt	Cairo	A6
Equatorial Guinea	Malabo	A9
Eritrea	Asmara	A8

Country	City	Location Code
Ethiopia	Addis Ababa	A8
Gabon	Libreville	A9
Gambia	Banjul	A8
Ghana	Accra	A9
Guinea	Conakry	A8
Guinea-Bissau	Bissau	A8
Iran	Tehran	A5
	Shiraz	A6
Iraq	Baghdad	A5
Israel	Jerusalem	A6
Jordan	Amman	A6
Kenya	Nairobi	A9
Kuwait	Kuwait	A6
Lebanon	Beirut	A5
Lesotho	Maseru	A6
Liberia	Monrovia	A9
Libya	Tripoli	A6
Madagascar	Antananarivo	A8
Malawi	Lilongwe	A8
Mali	Bamako	A8
Mauritania	Nouakchott	A8
Mauritius	Port Louis	A7
Morocco	Rabat	A5
Mozambique	Maputo	A7
Namibia	Windhoek	A7
Niger	Niamey	A8
Nigeria	Abuja, Lagos	A8
Oman	Muscat	A7
Qatar	Doha	A7
Rwanda	Kigali	A9
Sao Tome and Principe	Sao Tome	A9
Saudi Arabia	Riyadh	A7
Senegal	Dakar	A8
Seychelles	Victoria	A9
Sierra Leone	Freetown	A8
Somalia	Mogadishu	A9
South Africa	Cape Town	A5
	Durban	A6
	Pretoria	A7
South Sudan	Juva	A9
Sudan	Khartoum	A8
Swaziland	Mbabane	A7
Syria	Damascus	A5
Tanzania	Dar es Salaam	A8
Togo	Lomé	A9
Tunisia	Tunis	A5
Turkey	Ankara, Bursa, Istanbul	A4
	Adana, Izmir	A5
Uganda	Kampala	A9
United Arab Emirates	Abu Dhabi	A7
Yemen	Sanaa	A8
Zambia	Lusaka	A8
Zimbabwe	Harare	A8