


## Installation of the TANITA Pro software

---

Click this link to download the most recent version of the software:

[https://www.gmon.eu/download/pro/GMonPro\\_348\\_hgzus](https://www.gmon.eu/download/pro/GMonPro_348_hgzus)


The download will start automatically, you will see this in the left bottom corner of your computer:

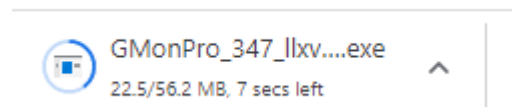

When it's done downloading, click on it.

- If asked, you'll need to enter the Administrator password of your computer now. If you don't have this password, contact your IT department.


This screen will appear, just wait for it to finish:

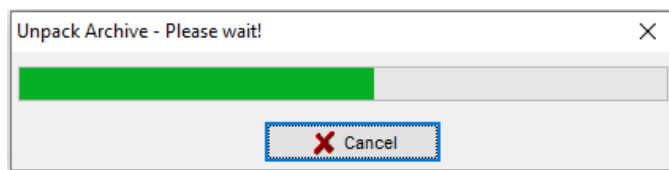


When it's finished, this screen will appear, click 'Next':




Accept the license agreement by checking the box, and click 'Next':

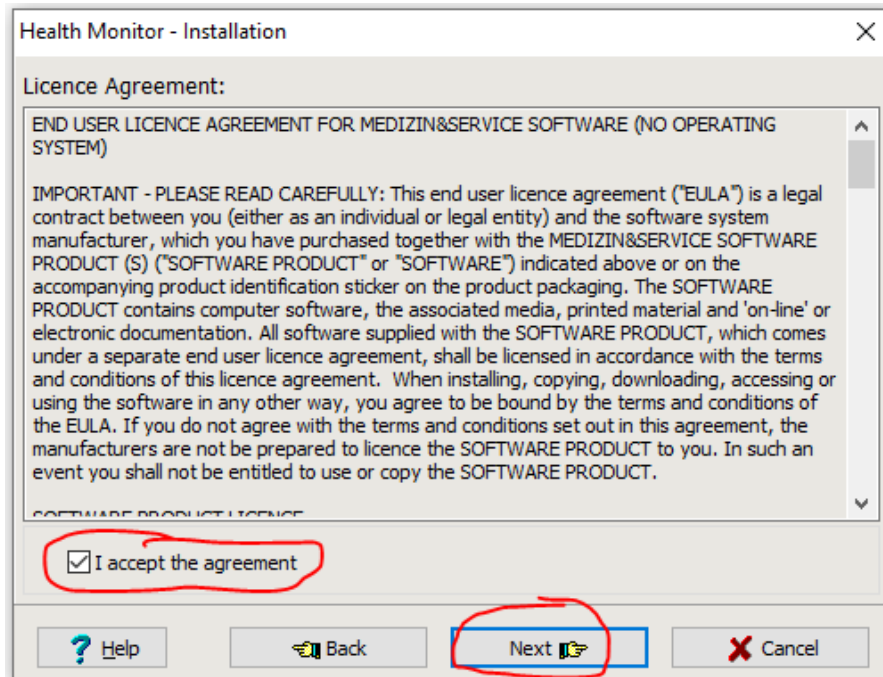

Check BOTH boxes, leave the rest as it is, and then click 'Next':  
(In this example I cannot check the 'Install FireBird Database' box, because it's already installed.)

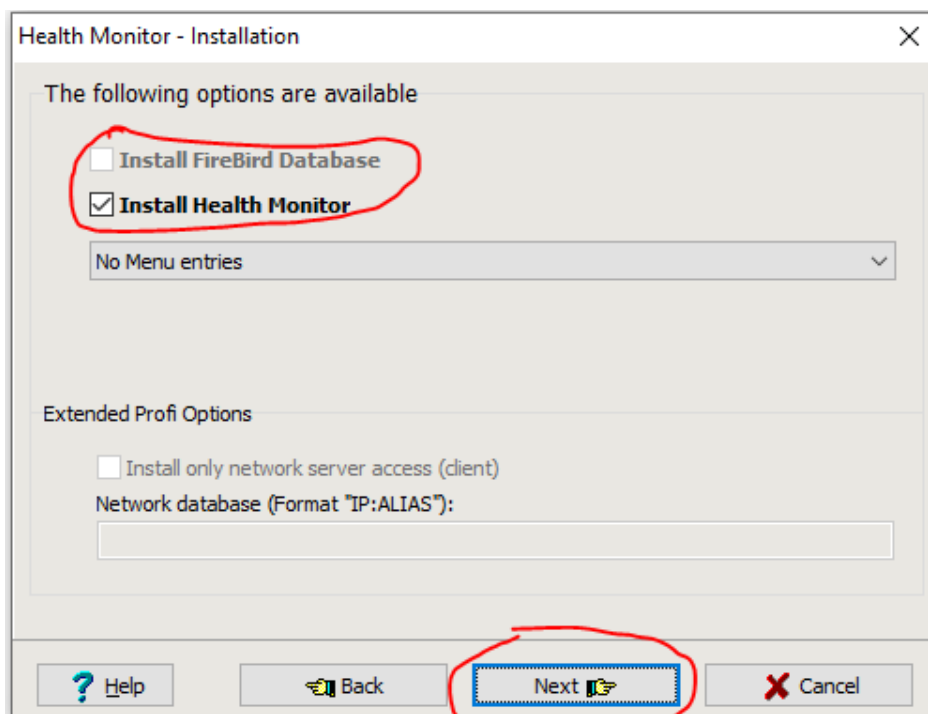


Enter the SNR of the software in the top field, and the LIC in the bottom field and click 'Next':

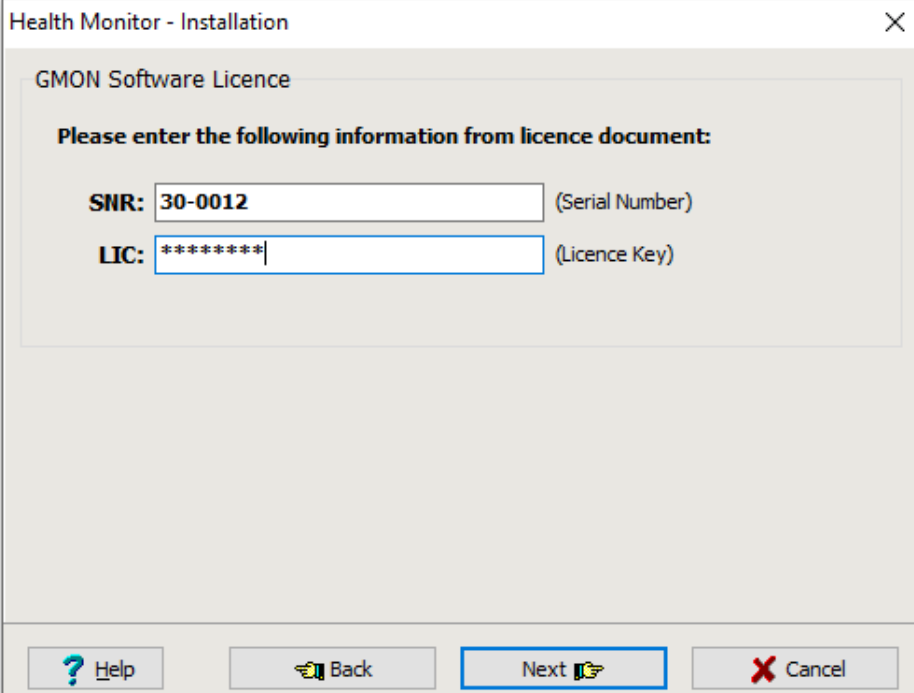

The screenshot shows a window titled "Health Monitor - Installation". Inside, there's a section for "GMON Software Licence" with the instruction "Please enter the following information from licence document:". Below this, there are two input fields: "SNR:" with the value "30-0012" and "(Serial Number)" to its right, and "LIC:" with the value "\*\*\*\*\*" and "(Licence Key)" to its right. At the bottom, there are four buttons: "Help" (with a question mark icon), "Back" (with a left arrow icon), "Next" (with a right arrow icon and highlighted with a blue border), and "Cancel" (with a red X icon).


The SNR always starts with '30-' and the LIC has a format somewhat similar to '0AB-1C2-DE'.

Leave everything like it is and click on 'Update' or 'Install':

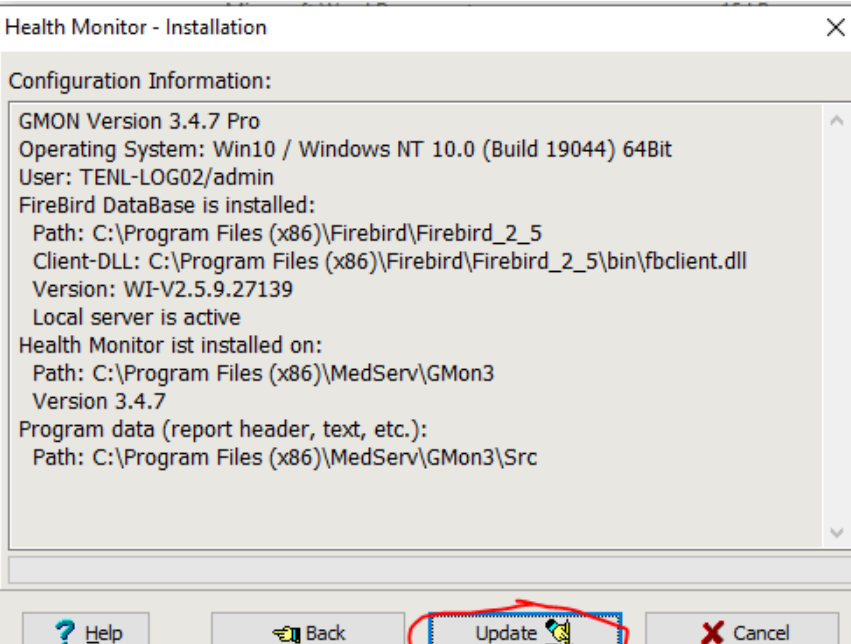


The screenshot shows the same "Health Monitor - Installation" window, but now displaying "Configuration Information:". The text inside lists the following details: "GMON Version 3.4.7 Pro", "Operating System: Win10 / Windows NT 10.0 (Build 19044) 64Bit", "User: TENL-LOG02/admin", "FireBird DataBase is installed:" followed by its path, client-DLL, and version, "Local server is active", "Health Monitor ist installed on:" followed by its path and version, and "Program data (report header, text, etc.):" followed by its path. At the bottom, the buttons are "Help", "Back", "Update" (with a right arrow icon and circled in red), and "Cancel".


The software will now install:

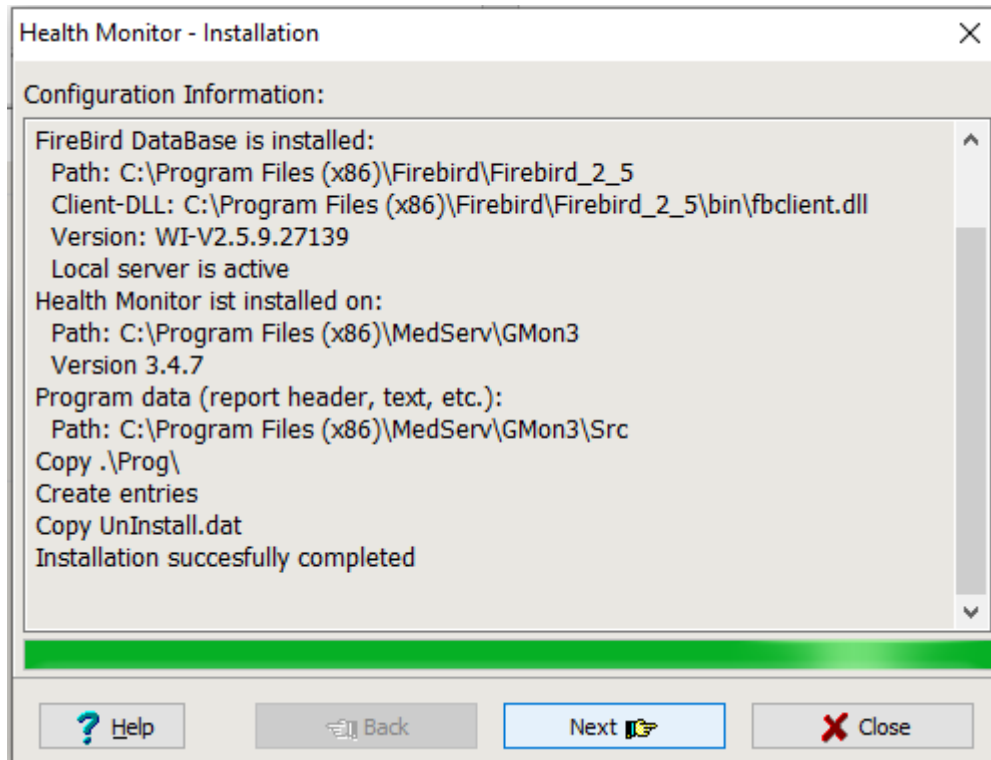

When it's done, you'll see this screen:

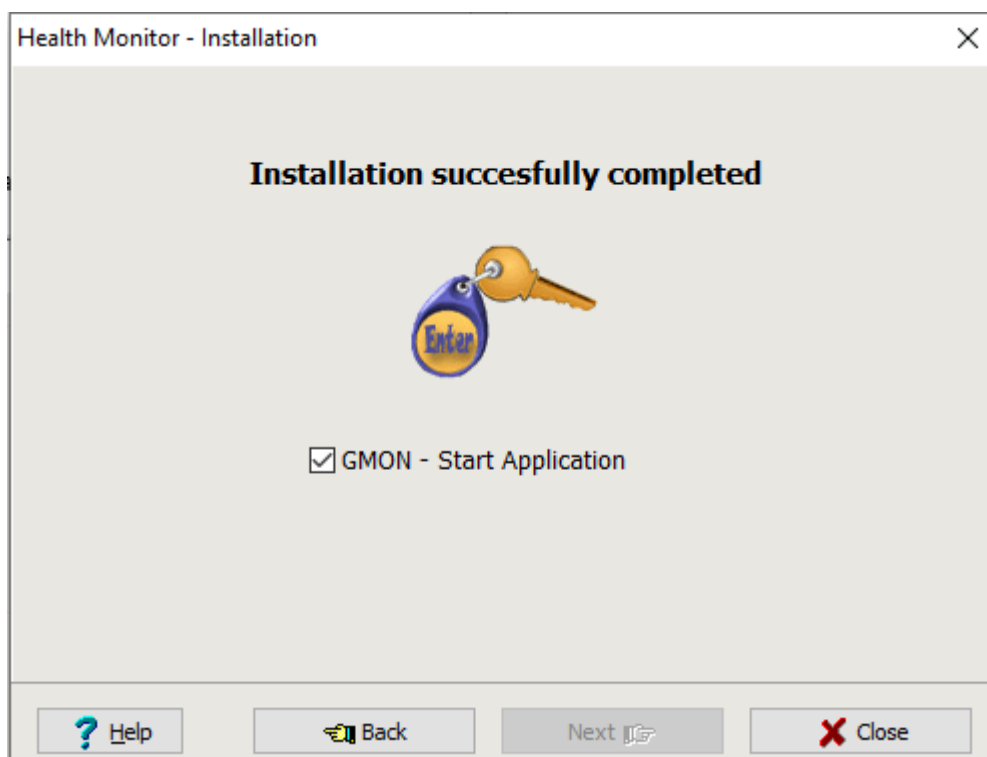

After pressing 'Close', the Customer Administration will open.  
Now you can proceed by importing the database of which you previously made a backup from your old computer, or you can start by creating a new profile.

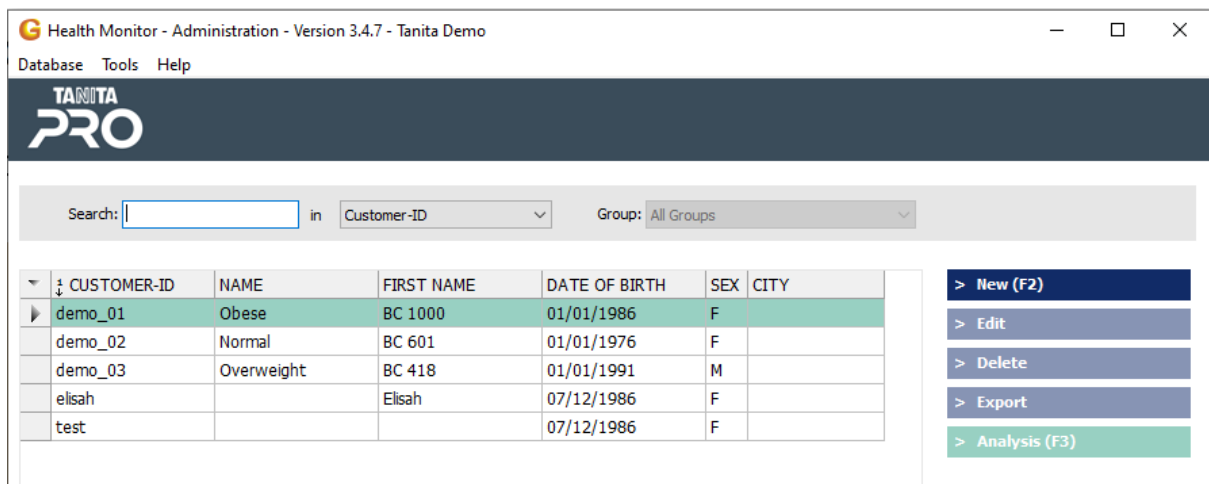

Connect your TANITA device with a standard printer cable (USB A <> USB B), in this position:



When creating a new profile, fill in this information:  
(The field "Customer-ID" can be left blank; the ID will then be generated automatically.)

After creating or selecting a customer, click "New measurement":

The following parameters must be set for the very first measurement:  
(When your scale is another type, choose another type from the drop-down menu.)

The screenshot shows the 'Tanita Scale' software window. It has a title bar with a close button (X) in the top right corner, indicated by a red arrow. The window is divided into several sections. On the left, under 'Values', there are three input fields: 'Body Type' (set to 'Normal'), 'Body Height [cm]' (set to '160', circled in red), and 'Clothes Weight [kg]' (set to '0,0'). Below these is a checkbox for 'Weight Only'. On the right, under 'Parameters for Scale', there are four dropdown menus: 'Type' (set to 'MC-780'), 'Mode' (set to 'Remote Mode'), 'Print' (set to '0x' and 'German'), and 'Interface' (set to 'USB1'). This entire 'Parameters for Scale' section is circled in red. At the bottom left, there is a status bar that says 'Interface USB1 OK'. Below the status bar are three buttons: 'Start Measure' (with a green checkmark icon, circled in red), 'Cancel' (with a red X icon), and 'Help' (with a question mark icon). On the far right, there is a small image of the Tanita MC-780 scale device.

Make sure that your TANITA device is switched ON (and is on PC-mode), that the cable is inserted in the device and in the computer and then press 'Start Measure' to initiate a measurement.

Follow the instructions and allow the measurement to finish.  
When finished, the results will be transferred to the software, where you can save the results.