

## TANITA PRO software


Settings for Gmail to send emails with reports to customers.

To give TANITA PRO software access to send emails from your Gmail account, you need to change some security settings in your Gmail account.


Here you can find a step-by-step guide.

---


1. Log in into your Gmail account.
2. Go to 'Manage your Google account' and click on it:

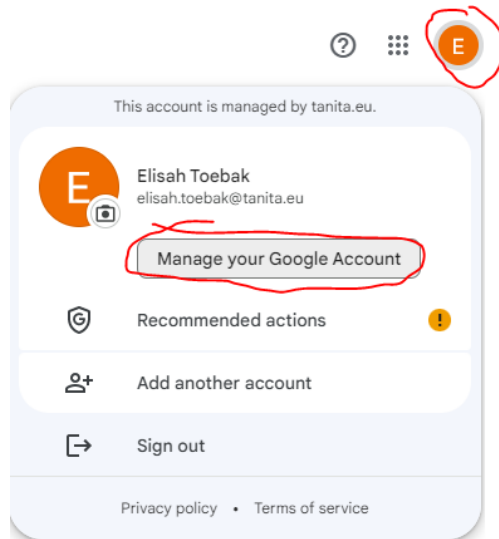

3. Click on 'Security' on the left:

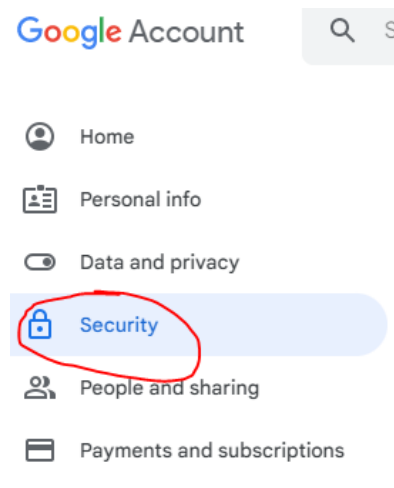

4. Scroll down to 'How you sign in to Google' and click on 2-Step Verification:

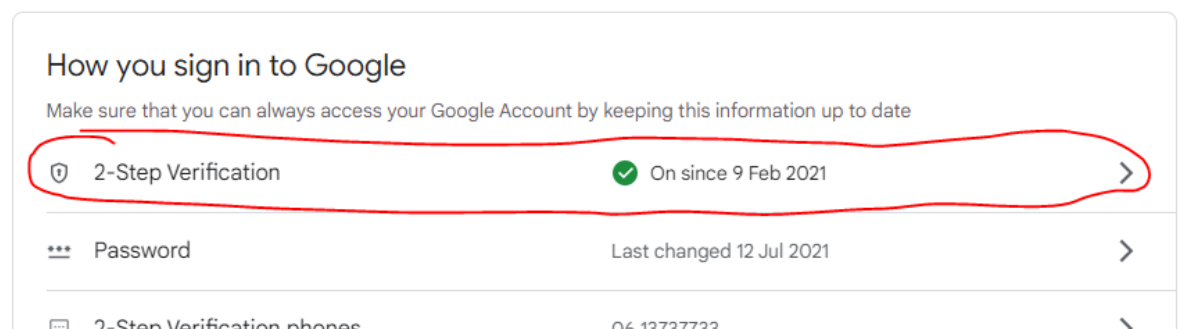

## TANITA PRO software

Settings for Gmail to send emails with reports to customers.

5. Enter your password to continue.
6. Scroll down to 'App passwords' and click on that:

### App passwords


App passwords aren't recommended and are unnecessary in most cases. To help keep your account secure, use 'Sign in with Google' to connect apps to your Google Account.



7. Select 'Mail' in the drop-down menu on the left, select your device on drop down menu on the right (likely Windows Computer):
8. Click on 'Generate':

### ← App passwords

App passwords let you sign in to your Google Account from apps on devices that don't support 2-Step Verification. You'll only need to enter it once so you don't need to remember it. [Learn more](#)

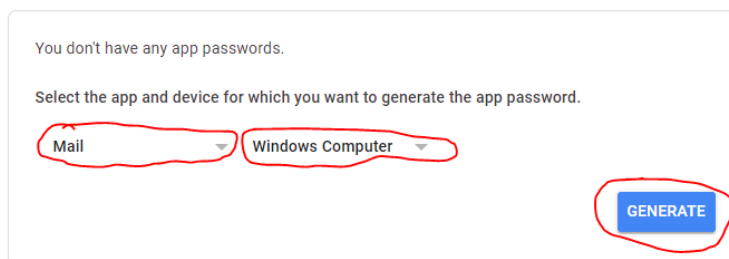

9. A new display will open, with a 16-character password in yellow. Copy that password, and use that in the TANITA PRO email account setup:

### Generated app password

Your app password for Windows Computer




#### How to use it

1. Open the "Mail" app.
2. Open the "Settings" menu.
3. Select "Accounts" and then select your Google Account.
4. Replace your password with the 16-character password shown above.

Just like your normal password, this app password grants complete access to your Google Account. You won't need to remember it, so don't write it down or share it with anyone.

[Learn more](#)

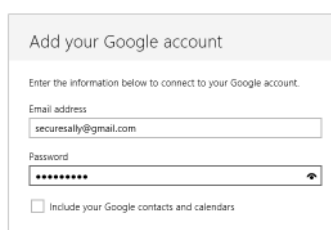

DONE